


Instructions for fitting a HELIX™ filter into a HELIX.iso™ mask body

1. Wash your hands with soap and water or use hand sanitiser.

2. Take one HELIX.iso™ mask body and one Helix™ filter.



3. Open the mask body and begin inserting the filter. The filter should be inserted so the filter “peak” is at the nose-band end of the mask body.

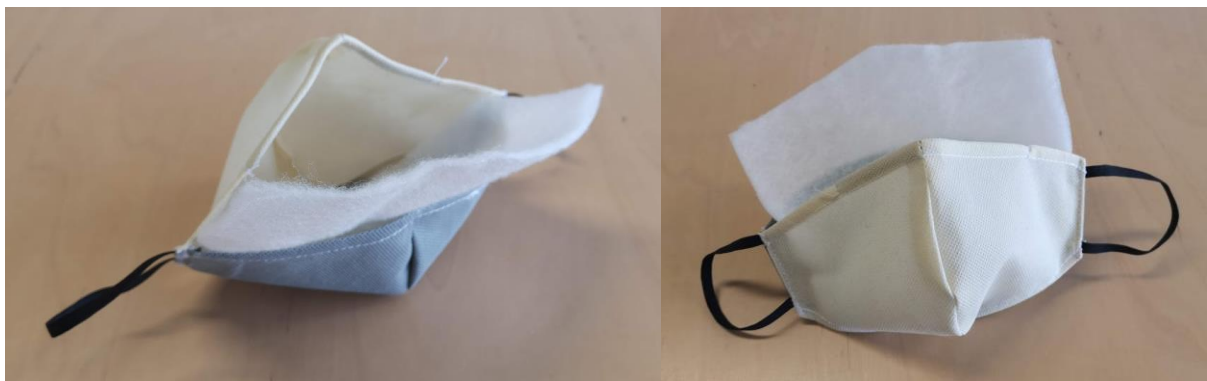

4. Continue inserting the filter so the two bottom corners protrude out of either side at the bottom of the mask.



5. Close the mask body and ensure the filter is still positioned correctly. This includes checking that:
 - a. The bottom corners of the filter are protruding out at the bottom of the mask body.
 - b. The top edge of the filter is just visible along the top edge of the mask body. This ensures a better seal along the nose bridge region.
 - c. The sides of the filter run along the sides of the mask body.

